

By your side
and beyond



Cardinal Health™ Kangaroo OMNI™ Enteral Feeding Pump


Cleaning and disinfecting guide



It is recommended that the pump be cleaned for a minimum of 30 seconds after each feeding-set use to prevent bacterial contamination. Failure to clean the pump can interfere with the pump rotor and sensor, which can increase the occurrence of errors and warning alarms.




Inspect the device for any visible soil immediately following usage and/or procedure. If visible soil is present, wipe the pump with a paper towel moistened with a 20:1 ratio of water-to-detergent mixture, removing all visible soil. Use a brush to remove soil from hard to reach crevices.

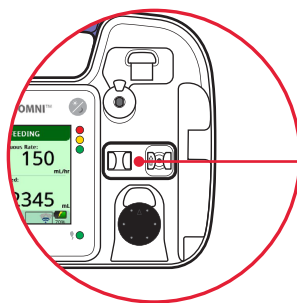

To disinfect the Kangaroo OMNI™ Enteral Feeding Pump, use at least two 70% isopropyl alcohol, lint-free wipes or wipes saturated with 10% bleach solution as necessary to maintain visual wetness.

Gently clean and dry the occlusion sensor region.

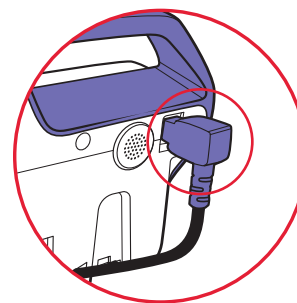

CAUTION: The power adapter will be damaged if cleaned with a wet cloth, placed under running water or submerged in liquid. If cleaning is necessary, unplug from the outlet and wipe the exterior surfaces of the AC adapter with a non-abrasive dry or slightly damp cloth. Allow excess moisture to evaporate prior to use.

For more information, please refer to the Kangaroo OMNI™ Enteral Feeding Pump Instructions for Use or online at mykangaroolearning.com

